




4 Easy Steps!



(1) Collect Specimen

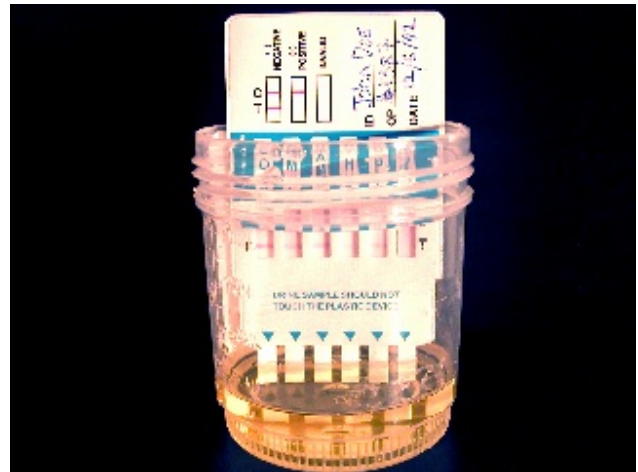

(2) Remove Cap & Dip Test Strips for 30 Seconds


- Negative results may be read in one to two minutes.
- Positive results may be read in three to five minutes.
- Tests which do not produce a Control line are invalid.
- Line intensity may vary from drug to drug however, ANY line in the Test zone should be regarded as a Negative result.

Amphetamines (AMP)	Methadone (MTD)
Barbiturates (BAR)	Methamphetamines (mAMP)
Benzodiazepines (BZO)	Opiates (OPI)
Cocaine (COC)	Opiates 300 (MOP)
Ecstasy (MDMA)	PCP (PCP)
Marijuana (THC)	Tricyclic (TCA)

(3) Return the Cap & Wait 3 Minutes



(4) Read Test Lines for Results

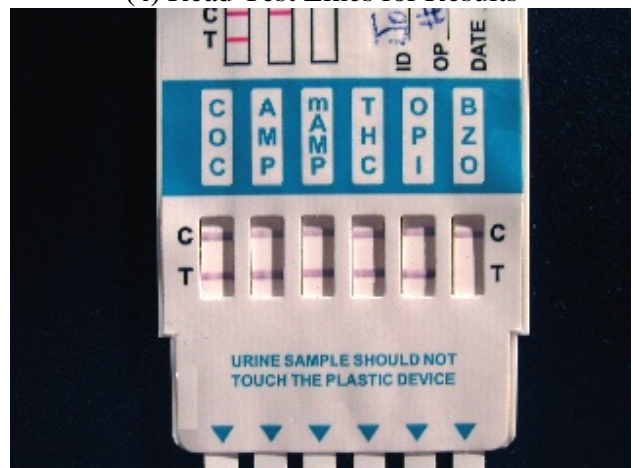

(two lines = negative, one line = positive)

Instant Drug Screens provide only a qualitative preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. See package insert for more details.

QuikTest USA, Inc. – 10959 Handel Place Boca Raton, FL 33498 – (866) 989-9300 – www.quiktestusa.com